


## Rabin Cone Test - Patient Orientation

1. Wear your glasses or contact lenses needed to best see at the test distance. This may require your readers. If you do not have your glasses with you or if you have blue-blocking lenses, ask the technician for a trial pair.
2. A single colored letter will appear in the center of the screen. Your task is to use the mouse or touch-screen to click on (select) the letter you see from the matching display on the lower right part of the screen. In the example shown, you would click-on "H". If you have trouble using the mouse or touch-screen, ask your technician to enter your response for you. Once you select a letter the computer repeats your response aloud.

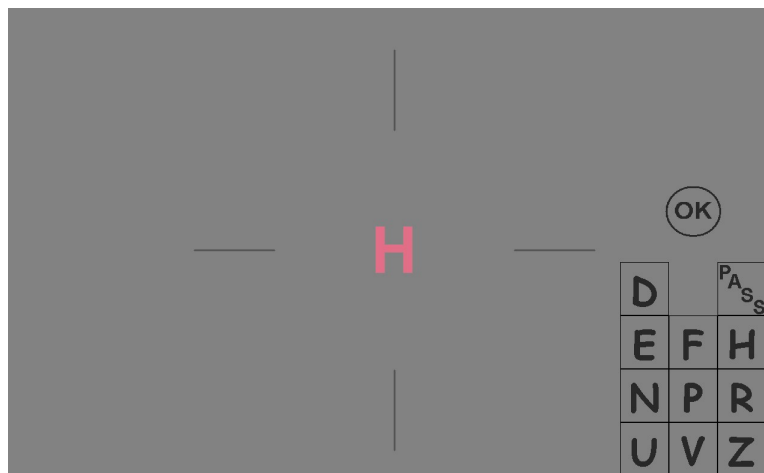

3. The colored letters became fainter and harder to see during the test. Some of the letters you may not be able to see. Do your best to select the letter you think you see, or you may select PASS. The test continues until it measures the faintest red, green and blue letters you are able to see. Each eye is tested independently.
4. The lights will be turned down while you are taking the test. If any glare from a light source interferes with your seeing the letter, make sure to tell the technician.
5. You may practice before beginning the test. Once you begin the test, it should take approximately 3-5 minutes to complete.